



!THIS IS YOUR NEW “WAX TRY-IN” Denture Prototype!
If fit and function are good, the patient can keep the Prototype!

Digital Dentures Instruction Sheet

(see reverse side for info)

407.447.4250 3501 Parkway Center Ct. Orlando FL 32808

- If fit is off, please make adjustments and take a wash impression
- If bite is off, please make adjustments and take a blue bite
- If midline or smile line is off, with the denture seated in the mouth, mark the midline and or lip line with a sharpie

All prototypes will be Vita A1 shade. Your final denture will be sent with the shade specified on the final RX. Must be a Vita Shade

After the Try-in is done please do the following:

Go to our website and download an Rx, fill it out with complete instructions on what you want to do next.

Email the new Rx with specific instructions and photos to:
Info@GPSDental.com **OR** send the prototype back to the lab with
the new wash, and or bite